

Launch of new iBGStar® Diabetes Manager Application v. 2.1.1 with iPhone 5 /iPod touch 5th Generation compatibility

Sanofi is pleased to announce the release of the new iBGStar® Diabetes Manager Application (DMA) v. 2.1.1 that is now compatible with the Apple iPhone 5 and Apple iPod touch 5th Generation devices via Lightning-to-30pin adaptors (sold separately).

New DMA v. 2.1.1

The new DMA v. 2.1.1 is available free of charge for download (or update) via the Australian iTunes App Store from 14th December 2012.

iBGStar® DMA v. 2.1.1 is safe for use with the new iPhone 5 and iPod touch 5th Generation and, as well as with iPhone 3G, 3GS, 4 and 4S, and iPod touch 2nd, 3rd and 4th Generations.

iBGStar® safe for use with both Lightning-to-30pin adapters

Sanofi has tested the two Lightning-to-30pin adapters released by Apple, both the [standard adapter](#) & [0.2m cable adapter](#), and can confirm the iBGStar® is safe for use with the iPhone 5 and iPod touch 5th Generation with these adapters.



No other adapters have been tested at this stage, and safety of iBGStar® in conjunction with iPhone 5 /iPod touch 5th Generation + non-Apple adapters cannot be confirmed. As a consequence, Sanofi does not currently recommend the use of non-Apple adapters in conjunction with iBGStar®. iPhone 5 /iPod touch 5th Generation users are advised to use the original Apple adapter.

We'd like to remind you that iBGStar® is also still functional as a stand-alone blood glucose meter (without being connected to your iOS device) and can store up to 300 test results. If you choose to use your older generation iPhone, even if you don't use it as a phone with a calling plan, iBGStar® will still retain full functionality and sync your results when connected.

BGStar[®] DMA Cable safe for use with both Lightning-to-30pin adapters

We have also tested the use of BGStar[®] DMA cable (BGStar[®] + cable + adapter + iPhone 5 / iPod touch 5th Generation) and can confirm that it is safe to use BGStar[®] with the new iDevices through the BGStar[®] DMA cable (pictured below) and adapters developed by Apple.



No other adapters have been tested at this stage, and Sanofi does not recommend the use of non-Apple adapters in conjunction with BGStar[®] DMA cable. The BGStar[®] DMA cable is available online from www.bgstarwebshop.com.au.

For further information, please contact the STARcare customer support on 1800 BGSTAR (1800 247 827), visit www.bgstar.com.au or ask our Diabetes Support Partners.

iBGStar[®] and BGStar[®] are designed to assist people living with diabetes to monitor their blood glucose levels. Use only as directed. Consult your healthcare professional for advice.

The iBGStar[®] does not include the iPhone, iPod touch or Lightning-to-30pin adaptor. iBGStar[®] Diabetes Manager App is available free on the iTunes App store.

iPhone, iPod touch and iTunes are trademarks of Apple Inc., registered in the US and other countries.

**sanofi-aventis Australia Pty Ltd trading as Sanofi
Building D 12-24 Talavera Road Macquarie Park NSW 2113 Australia
AU.BGM.12.12.001**